

How to Print and Use the CoolSculpting® Applicator Templates

This file contains 4 CoolSculpting applicator templates (located on pages 2 through 4):

CoolSmooth PRO™ • CoolAdvantage™ • CoolAdvantage Plus™ • CoolAdvantage Petite™

Step 1: Print the applicator templates in this file and have them ready to use during your virtual consultation.

Printing Tips:

- Print each applicator template on an 8.5"x 11" white sheet of paper
- Print each template "actual size" and "one-sided" (some printers reduce the size to fit within a border and/or are set to print on both sides)
- Cut each printed template out along the outside edge

CoolSculpting applicator templates are for virtual assessments only.

Step 2: Your provider will tell you how and where to place each applicator template to help determine your CoolSculpting treatment plan.

- Please note that your treatment plan may change when your provider performs an in-office physical assessment

If you are unable to print the applicator templates, please have a 12" ruler or measuring tape ready to use during your virtual consultation.

If you have any questions or need additional help, please reach out to your provider before your virtual consultation.

Uses

The CoolSculpting® procedure is FDA-cleared for the treatment of visible fat bulges in the submental (under the chin) and submandibular (under the jawline) areas, thigh, abdomen and flank, along with bra fat, back fat, underneath the buttocks (also known as banana roll) and upper arm. It is also FDA-cleared to affect the appearance of lax tissue with submental area treatments. The CoolSculpting® procedure is not a treatment for weight loss.

Important Safety Information

The CoolSculpting® procedure is not for everyone. You should not have the CoolSculpting® procedure if you suffer from cryoglobulinemia, cold agglutinin disease, or paroxysmal cold hemoglobinuria.

Tell your doctor if you have any medical conditions including recent surgery, pre-existing hernia, and any known sensitivities or allergies.

During the procedure you may experience sensations of pulling, tugging, mild pinching, intense cold, tingling, stinging, aching, and cramping at the treatment site. These sensations subside as the area becomes numb. Following the procedure, typical side effects include temporary redness, swelling, blanching, bruising, firmness, tingling, stinging, tenderness, cramping, aching, itching, or skin sensitivity, and sensation of fullness in the back of the throat after submental or submandibular area treatment.

Rare side effects may also occur. CoolSculpting® may cause a visible enlargement in the treated area which may develop two to five months after treatment and requires surgical intervention for correction.

Please see full Important Safety Information on coolsculpting.com.

CoolSculpting® Applicator Templates

CoolSculpting applicator
templates are for
virtual assessments only.



CoolSculpting® Applicator Templates

CoolSculpting applicator templates are for virtual assessments only.



CoolSculpting® Applicator Templates

**CoolSculpting applicator
templates are for
virtual assessments only.**

